



AIRPRO MASK

BROAD Clean Air Product Air Pro Collection

Filter PM2.5 by 99.9%

Protection for renovation workers, welders and miners against silicosis, Prevention of cross-infection in hospitals, Protection for joggers against exhaust fumes.

Model: FB2










Auto particle-filtering respirator Standard: GB2626-2006 KN95



BROAD CLEAN AIR TECHNOLOGY 匠大洁净空气科技有限公司


Sales: 400-677-6666 Service: 400-883-9333 www.broadair.net BROAD Town, Changsha, China



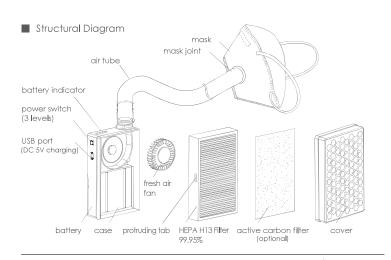

Standard Accessories



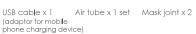











Lanyard x 1

Velcro x 1

■ Troubleshooting

No.	Problem	Cause	Solution
1	Insufficient air volume	The HEPA filter is blocked	Replace the HEPA filter
2	The fan does not work	The fan or circuit board is damaged, the battery is dead, or the battery separation film is not removed	Repair or replace it, charge the battery or remove the battery separation film
3	Charging failure	The circuit board or USB cable or adapter is damaged	Replace or repair it
4	The HEPA filter does not fit	The HEPA filter is backward	Put the protruding tab side upwards
5	The air tube is leaking	The air tube is aged or broken	Replace it

Parameter Table

Item	Level 1	Level 2	Level 3
Air Volume m³/h	2	3	4
Power W	0.6	0.9	1.3
Noise dB(A)	27	30	33
Battery	3.7V lith	ium batt	ery
Filtration Efficiency	Filter PN	12.5 by 99	9.9%

■ Air Volume Adjustment (press the power switch for 2 secs to turn on)

- · Remove the battery separation film before use for the first time
- · Level 1 and Level 2 are for daily use
- · Level 3 for physical labor and sports
- · Suggestion for first time wearing: run the AirPro for 1~2 hours to sufficiently clean out before wearing (with air tube and mask connected)

■ Mask Replacement

- Prepare a new mask and draw a □12~14mm circle on the left or right side near the nostril
- Fold the circle in half and cut the hole with scissors
- · Connect the mask to the mask joint and air tube

■ Regular Wearing Method

- · Attach the AirPro to the lanyard and hang it on your chest, put on the face mask with the air tube connected
- When the outdoor temperature is low, the main unit should be put in the pocket or hung on the chest to reduce the discomfort caused by cold air
- · You can also remove the lanyard from the Velcro and attach the Velcro to your helmet for easier work

■ HEPA Filter Replacement (after shutdown and power off)

• When the air volume is less than 70%, the HEPA

- filter needs to be replaced
- Replace the HEPA filter: Open the cover, take out and replace the HEPA filter. The HEPA filter should be replaced after 500 hours of use. When installing the HEPA filter, the protruding tab must point upward, otherwise the HEPA filter will not fit
- · Activated carbon is useful for absorbing toxic chemical fumes or gas, but it is very thin and easy to saturate. It should be replaced after 100 ~ 200 hours of use. For painters and renovation workers, replace every 2~5 days
- The mask needs to be cleaned or replaced if it is dirty

■ Warranty

- 1-year warranty from the date of purchase. For problems caused by design, manufacturing or transportation from factory, free repair or replacement will be provided
- · If the user is unable to provide a valid proof of purchase, the warranty period should be extended by 3 months from the date of production shown on the unit
- If the AirPro was damaged due to inappropriate use or storage, or the sticker was tom off, there will be a charge for repair

Maintenance

- Replace the HEPA filter when needed
- Keep the AirPro away from liquid, and avoid exposure to rain
- Avoid hitting, dropping or impacting the AirPro
- Design life: 20 years (the bearing may need to be replaced during this period)

Name:	AirPro Mask
Model:	FB2
Serial Nu	ımber:
Inspecto	or: QC
Date of	Purchase:
Shop:	
Seller:	